

Complete Application Instructions

SHEER WHITE!TM

Patent Pending

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed professional.

Disclaimer: CAO Group believes this information to be accurate and is offered only for the benefit of its customers for use of the product under proscribed conditions. This document is not to be considered a warranty or guarantee of product performance, and CAO Group is not legally bound to such claims based on this document.

Manufactured by CAO (China) Medical Equipment Co., Ltd. for



CAO GROUP, INC.
Easier · Faster · Better™

P 877.236.4408 F 801.256.9287

4628 West Skyhawk Drive, West Jordan, UT 84084-4501 U.S.A.

www.caogroup.com

Simple. Brite. White.®

20% Carbamide
Peroxide Whitening Film

Description

Sheer White!® whitening films are the most comfortable, patient friendly whitening system available today. These films are designed to adhere to teeth for 1 to 2 hours to take advantage of the proprietary carbamide peroxide composition.

PLEASE NOTE: removal before 30 minutes can be done, but is more difficult, may tear and come off in pieces.




Counter Indications

- Do not smoke during whitening treatment.
- Avoid contact with eyes. If contact occurs flush with copious amounts of water.
- Avoid direct contact of the active surface of the film with the gums and/or salivary flow.
- Not for use by women who are pregnant or nursing.
- Not for use by children under 18 years of age.
- Avoid contact with clothing.
- Do not swallow the film.
- If irritation (such as redness, swelling, soreness) of the gums or mouth occurs, discontinue use and consult a dentist.
- If you brux or grind your teeth, do not wear overnight.
- Do not use if allergic to peroxides like hydrogen peroxide.
- Do not use for more than 5 days per regimen.
- Whitening will not affect porcelain crowns and veneers tooth colored acrylic restorations or amalgam.

Storage & Handling

- Keep the product refrigerated to maintain maximum shelf life.
- Do not store above 40°F (5°C). Do not freeze. Best use before date on label.
- Do not use if the packaging has been damaged, or if the seals are found to be broken or open.
- 20% initial concentration of carbamide peroxide. The concentration may change due to the nature of peroxide and this unique formulation.

Shelf Life: 36 months


	Attention - Refer to the instructions for use and complete information.
	Oxidizer - Avoid contact with reactive substances.
	Storage Temperature - Store product at the indicated temperature.



1 The whitening side of the film faces UP. Grasp film at end and remove as shown.



2 Center film and place against teeth at gum line.

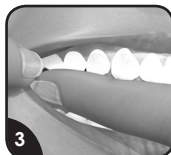

3 Pull cheek away and push film onto back teeth.



4 Fold film to back side of teeth. Press firmly on all surfaces.



5



6 To remove, grasp one corner and pull down away from front of teeth, then push film away from back of teeth.



7 Brush or rinse away any remaining gel. Rinse to sink.